



MEDVUE QUICKSTART GUIDE

ADJUST GAMMA LEVELS

- Activate control by pressing the spacebar or enter keys.
- Press the F4 key to display gamma control panel.
- Set desired gamma levels and click Apply.

START/STOP CINE DISPLAY

- Activate control by pressing the spacebar or enter keys.
- After cine display is loaded, left-click on gated image to pause; left-click again to resume motion.
- While images are paused, right-click to cycle frame by frame through cine images. Each click will advance by one frame.

BASIC SITE NAVIGATION

Select Patient

- After logging in, click on the patient name you wish to view. This displays a list of study images.
- Click any study image to view.

Cycle Through Images

- After selecting an image, use the Next and Previous buttons to cycle between images.

Return to Study List

- To return to the study list, click the Return button.

CHANGE COLOR PALETTES

Static Images

- Activate control by pressing the spacebar or enter keys.
- Left-Click on image to cycle through color palettes

Gated Images

- Activate control by pressing the spacebar or enter keys.
- Press Page-Up or Page-Down keys to cycle through color palettes.

ADJUST BRIGHTNESS

- Activate control by pressing the spacebar or enter keys.
- Click and hold Wheel button (requires a wheel mouse) and drag down to increase brightness; down to decrease brightness.

ADJUST CONTRAST

- Activate control by pressing the spacebar or enter keys.
- Click and hold Wheel button (requires a Wheel mouse) and drag right to decrease contrast; left to increase contrast.

FURTHER INFORMATION

- For detailed information on these and other functions, please see our Medvue User's Guide
- <http://www.frnsinc.com/Medvue/UserGuide/Medvue.aspx>